



# Needle and Syringe Programme – New Product Portfolio - Southampton

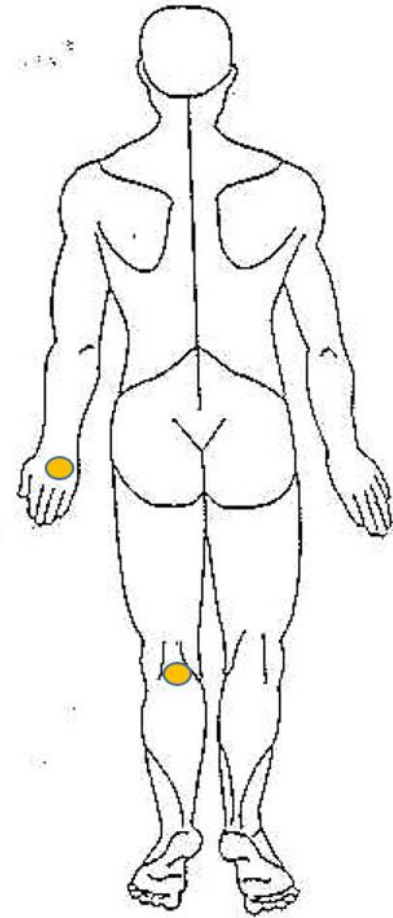

# Injecting Sites (Intravenous)

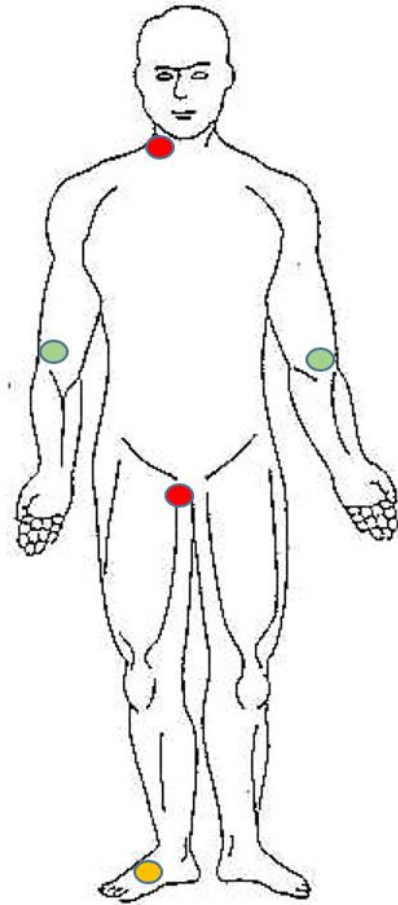

- Low risk
- Medium risk
- High risk

# 1ml 1 Hit Kit (019015 *citric*) 50 kits per bag

- 29g 1ml Fixed Syringe
- Prep spoon with filter
- Citric sachet
- Alcohol prep wipe
- **Suitable for small surface vein injecting (IV)**
- The syringe is also suitable for subcutaneous injecting (SC)
- This product is NOT suitable for intramuscular injecting (IM)





# 2ml 1 Hit Kit (blue)

**019027** citric (50 kits per bag)

- 23g 1 ¼" needle
- 2ml syringe
- Prep spoon with filter
- Citric
- Alcohol prep wipe
- Suitable for deep veins injecting e.g. groin (IV)
- The needle and syringe is also suitable for IM injecting (IM)



# 2ml 1 Hit Kit (orange)

**019028** citric (50 kits per bag)

- 25g 1" needle
- 2ml syringe
- Prep spoon with filter
- Citric
- Alcohol prep wipe
- Suitable for deeper veins injecting (IV)




The MySafe Plus is to be dispensed with the One Hit Kits.





Where clients are taking multiple OHKs (4 or more) offer a larger bin

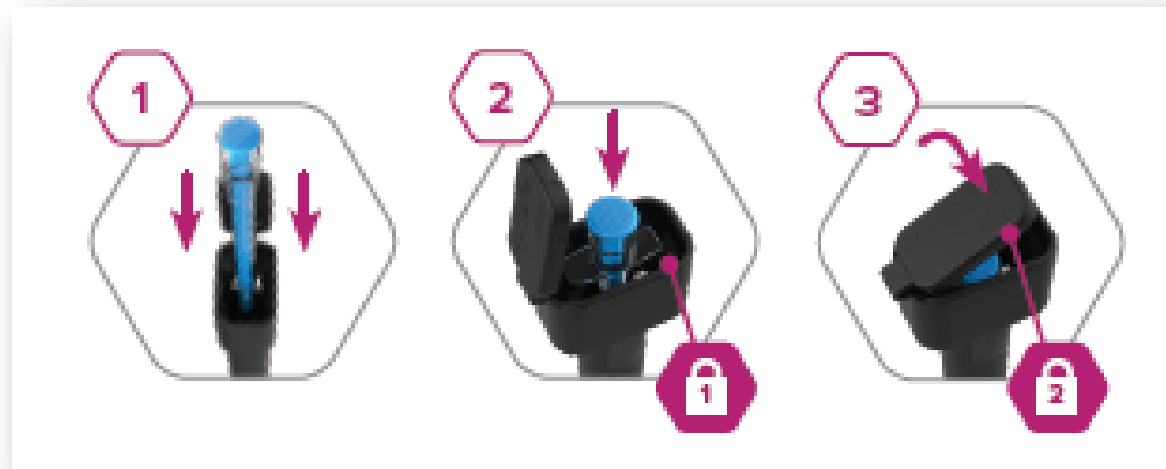Code: 20000002



# Ensure you offer a sharps bin with every NX transaction

Size	Approximate number of needles held	Useful Info
<b>0.2L</b> 	5 - 10 fixed 1mls	Temporary and final locking mechanism – push flap down to cover, fully depress for locking. <b>Useful for outside injectors</b>
<b>0.45L</b> 	10 - 12 fixed 1mls or 10 2ml barrels and needles if detached (using mechanism on bin)	Temporary and final locking mechanism – push flap down to cover, fully depress for locking. <b>Suitable for home use</b>



# X704 Steroid pack content (50 packs per carton)


---

- 10 x 2ml syringe
  - 10 x 23g 1 ¼" blue needle
  - 10 x 21g 1 ½" green needle (for drawing up)
  - 10 x swabs
  - 1 x 0.45ltr bin
- 
- Suitable for Intramuscular (IM) injecting



# Intramuscular Pack

- Includes enough equipment for 10 injections
- Suitable for Intra **Muscular injection (IM)**

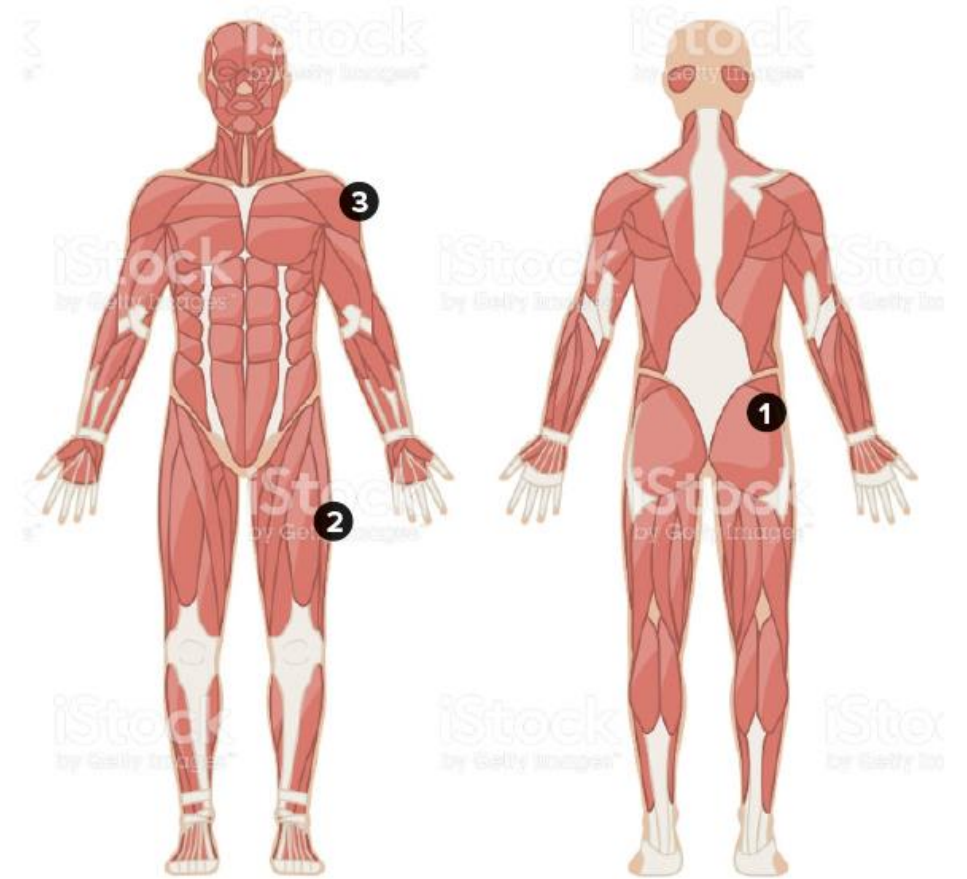

1. The upper outer quadrant of the buttock (low risk)
2. Lateral part of thigh (low/medium risk)
3. Deltoid-upper outer third of the shoulder region (medium/high risk)

# Placing orders

<https://fcom.uk/login>

- Using the link enter your username and password (you will have received an email with this detail)
  - Click 'Order' tab on left hand side
  - Click 'Create order'
  - Click 'Add item'
  - Select the items you require
  - Click 'Place order'
- 
- If you have any problems logging in contact: [NXSales@vernagroup.com](mailto:NXSales@vernagroup.com) or T: 01495 235 800

# For further information contact:

- Andrea Mulligan UK Harm Reduction Manager
- [Andrea.mulligan@vernagroup.com](mailto:Andrea.mulligan@vernagroup.com)
- Mob: +44 7980303814
- E-learning: <https://www.harmreduction.co.uk/solutions/elearning/>
- Online resources:  
IPEdDs (sports) [www.ipedinfo.co.uk](http://www.ipedinfo.co.uk)